



## Model 20000 Endoscope Storage Cabinet

### Installation Requirements

**Electrical Rating: 120 VAC, 5A, 60 Hz, 1/8 HP**

**Warning:** Ensure the cabinet's electrical plug is inserted into a properly grounded and dedicated, 120 VAC, 15-20A, 60 Hz, 1 Ph receptacle. Receptacle must be accessible to the operator for quick disconnect.

### Mechanical

Secure the cabinet flush to the wall with the weight of the cabinet resting on the floor using the mounting holes provided, along with materials suitable to facility's structure and local building codes. Allow a minimum 1" clearance above the exhaust vent output on top of the unit.

**Warning:** A minimum of (2) persons is required to support and mount the cabinet or bodily injury and physical damage to the equipment and surrounding area may occur.

